medi GmbH & Co. KG  
Medicusstraße 1  
D-95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-783  
ortho@medi.de  
www.medi.de

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium bvba  
Posthoornstraat 13/1  
3582 Koersel  
Belgium  
T +32 011 24 25 60  
F +32 011 24 25 64  
info@medi-belgium.be  
www.medi-belgium.be

medi Brasil  
Rua Neusa 216  
Diadema-Sao Paulo  
Cep 09941-640  
Brazil  
T +55 11 3201 1188  
F +55 11 3201 1185  
sac@medibrasil.com  
www.medibrasil.com

medi Danmark ApS  
Veilegardsvej 59  
2665 Vallensbæk Strand  
Denmark  
T +45 46 55 75 69  
F +45 70 25 56 20  
kundeservice@sw.dk  
www.meditandmark.dk

medi Bayreuth España SL  
C/Canigó 2 – 6 bajos  
Hospital de Llobregat  
08901 Barcelona  
Spain  
T +34 93 260 04 00  
F +34 93 260 23 14  
medi@mediespana.com  
www.mediespana.com

medi France  
Z.I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
T +33 1 48 61 76 10  
F +33 1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Hungary Kft.  
Bokor u. 21.  
1037 Budapest  
Hungary  
T +36 1371 0090  
F +36 1371 0091  
info@medi-hungary.hu  
www.medi-hungary.hu

medi Italia S.r.l.  
Via Guido Rossa 28  
40033 Casalecchio di Reno-  
Bologna  
Italy  
T +39 51 613 24 84  
F +39 51 613 29 56  
info@medi-italia.it  
www.medi-italia.it

medi Medical Support Sdn Bhd  
medi representative office Asia  
Unit No. B-2-19, Block B, No.2,  
Jalan PUJ 1A/7A  
Oasis Ara Damansara, PUJ 1A,  
47301 PETALING JAYA  
Selangor Darul Ehsan  
Malaysia  
T: +6 03 7832 3591  
F: +6 03 7832 3921  
info@medi-asia.com  
www.medi-asia.com

medi Middle East  
P.O. Box: 109307  
Abu Dhabi  
United Arab Emirates  
T +971 2 6429201  
F +971 2 6429070  
info@mediuae.ae  
www.mediuae.ae

medi Nederland BV  
Heusing 5  
4817 ZB Breda  
The Netherlands  
T +31 76 57 22 555  
F +31 76 57 22 565  
info@medi.nl  
www.medi.nl

medi Polska Sp. z o.o.  
Zygmunta Starego 26  
44-100 Gliwice  
Poland  
T +48 32 230 60 21  
F +48 32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

medi

## Knöchelbandage

Gebrauchsanweisung. Instructions for use. Mode d'emploi.  
Instrucciones de uso. Instruções para aplicação. Istruzioni per  
l'uso. Gebruiksaanwijzing. Brugsvejledning. Bruksanvisning.  
Návod k použití. Uputa za uporabu. Инструкция по  
использованию. Giynme kilavuzu. Instrukcja zakładania.  
Οδηγία εφαρμογής. Használati útmutató. Uputstvo za upotrebu.  
Інструкція з використання. Navodila za uporabo.

E006462/04.2014

medi. I feel better.

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### Important notes

This product is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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---

## **Intended purpose**

The product is exclusively to be used for the orthotic fitting of the ankle and only on unbroken skin.

---

## **Indications**

- Ankle sprains
- Swelling and joint effusion
- Rheumatic and osteoarthritic irritation

## **Contraindications**

Occlusive arterial disease, phlebitis, ulcers, after venous thrombosis; in case of varicosis and leg oedema a compression stocking up to the knee (A – D) should also be worn.

## **Applying the support**

Grasp the upper part of the ankle support (label) and pull it up over the ankle until the heel fits over the integrated recess.

---

## **Care instructions**

Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.

- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

### **Storage instructions**

Store in a dry place at room temperature. Protect from heat, direct sunlight and moisture.

### **Material composition**

Polyamide, Elastane

### **Warranty / contractual guarantee**

Consistent with legal guidelines.

### **Disposal**

The product can be disposed of in the domestic waste.

### **Your medi team**

wishes you a quick return to full fitness.

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